Bittium

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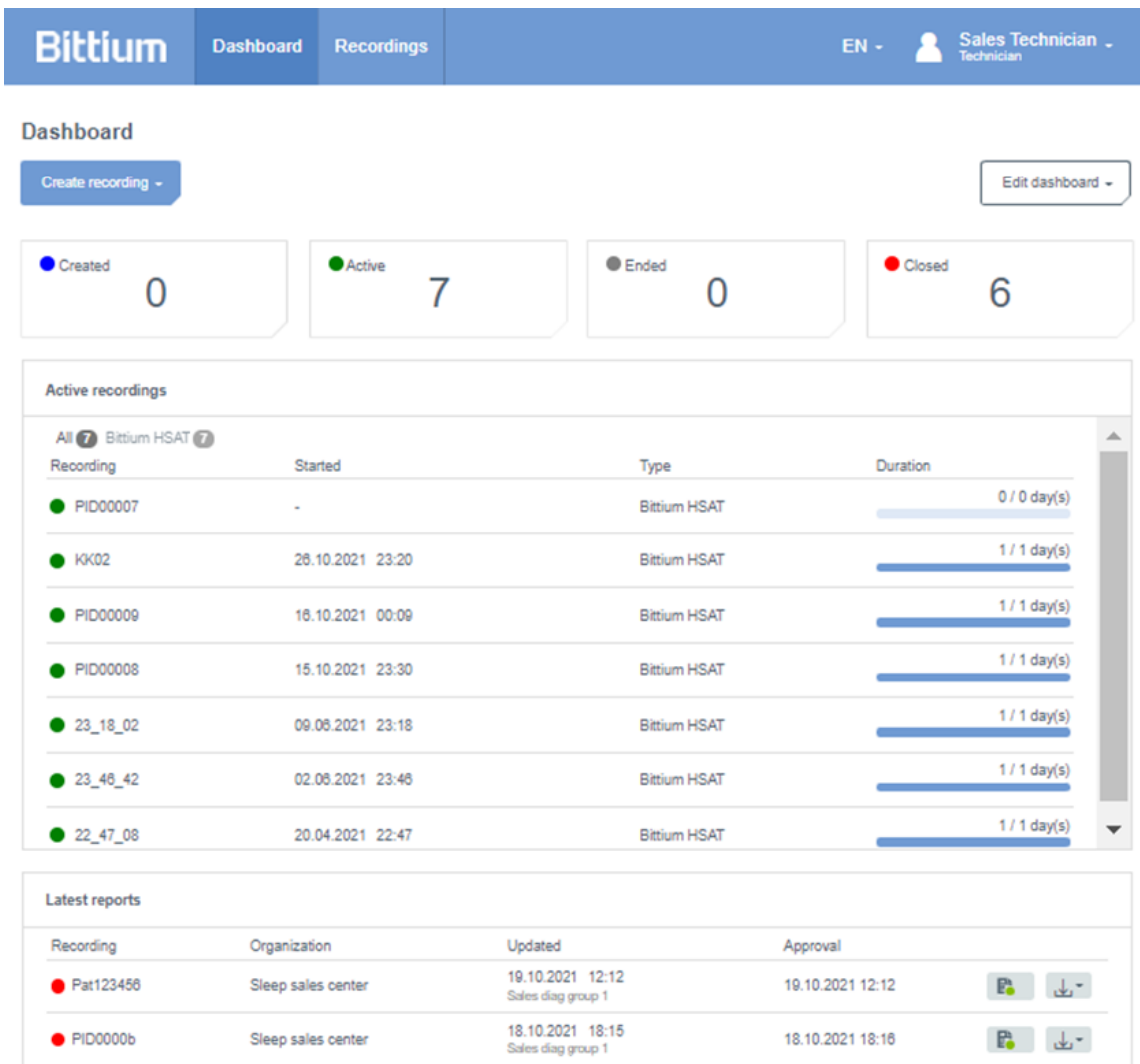
Illustration 8 MedicalSuite Center Dashboard-vy

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Recordings

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
KK02	Active	28.10.2021 15:25	28.10.2021 15:46	Sleep sales center	28.10.2021 15:23	Bittium HSAT	0000568	Sales diag group 1	Download
Pat123456	Closed	19.10.2021 11:41	19.10.2021 12:12	Sleep sales center	19.10.2021 11:40	Bittium HSAT	0000673	Sales diag group 1	Download
PID0000b	Closed	18.10.2021 17:48	18.10.2021 18:16	Sleep sales center	18.10.2021 17:47	Bittium HSAT	0000673	Sales diag group 1	Download
PID0000a	Closed	18.10.2021 16:19	18.10.2021 16:15	Sleep sales center	18.10.2021 16:18	Bittium HSAT	0000536	Sales diag group 1	Download
PID00009	Active	16.10.2021 08:30	09.11.2021 10:16	Sleep sales center	16.10.2021 08:29	Bittium HSAT	0000536	Sales diag group 1	Download
PID00008	Active	16.10.2021 08:25	16.10.2021 09:04	Sleep sales center	16.10.2021 08:25	Bittium HSAT	0000568	Sales diag group 1	Download
Test008	Closed	15.10.2021 19:01	18.10.2021 15:26	Sleep sales center	15.10.2021 19:00	Bittium HSAT	0000673	Sales diag group 1	Download
TEST007	Closed	15.10.2021 18:46	15.10.2021 19:14	Sleep sales center	15.10.2021 18:43	Bittium HSAT	0000673	Sales diag group 1	Download
PID00007	Active	No date	14.10.2021 17:22	Sleep sales center	14.10.2021 17:18	Bittium HSAT	1234567	Sales diag group 1	Download
23_45_08	Closed	01.10.2021 16:17	14.10.2021 17:19	Sleep sales center	01.10.2021 16:16	Bittium HSAT	0123456	Sales diag group 1	Download

Illustration 13 MedicalSuite Center-registreringsvyn

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The screenshot shows the 'Recordings' section of the Bittium interface. At the top, there are navigation tabs for 'Dashboard' and 'Recordings', along with user information for 'Technician 1'. Below the navigation is a search bar and a 'Create recording' button. The main area contains a table with columns for Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open for the 'Assignment' column of the first row, showing options like 'Cardioscope Group', 'Diagnostic group', and 'HSAT Analysis'.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
testPDF	Closed	23.01.2024 15:14	25.01.2024 13:41	ServiceProvider	23.01.2024 15:14	Bittium	0000683	HSAT Analysis	[Download]
HSAT_ECG	Ended	16.01.2024 16:13	16.01.2024 16:17	DemoClinic1	16.01.2024 16:13	Bittium			[Download]
HSAT_2_RIP_Belts	Active	16.01.2024 16:11	25.01.2024 10:10	DemoClinic1	16.01.2024 16:11	Bittium			[Download]
PID00314	Ended	14.12.2023 15:06	19.12.2023 13:09	DemoClinic1	14.12.2023 15:05	Bittium			[Download]
PID002f3	Closed	22.11.2023 11:37	22.11.2023 11:41	DemoClinic1	22.11.2023 11:37	Bittium			[Download]

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The screenshot shows the 'Recordings' section of the Bittium interface. At the top, there are navigation tabs for 'Dashboard' and 'Recordings', along with user information for 'Sales Technician'. Below the navigation is a search bar and a 'Create recording' button. The main area contains a table with columns for Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open for the 'Action' column of the first row, showing options like 'Recording data', 'Recording data 1 day(s)', 'Reports', and 'recording-information.pdf'.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
Test008	Closed	15.10.2021 19:01	18.10.2021 15:26	Sleep sales center	15.10.2021 19:00	Bittium HSAT	0000673	Sales diag group 1	[Download]
TEST007	Closed	15.10.2021 18:46	15.10.2021 19:14	Sleep sales center	15.10.2021 18:43	Bittium HSAT	0000673		[Download]
PID0000b	Closed	18.10.2021 17:48	18.10.2021 18:16	Sleep sales center	18.10.2021 17:47	Bittium HSAT	0000673		[Download]
PID0000a	Closed	18.10.2021 16:19	18.10.2021 18:15	Sleep sales center	18.10.2021 16:18	Bittium HSAT	0000576		[Download]

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< Back
Bittium
Dashboard
Recordings
EN ▾

 Sales Technician ▾
Technician

Patient ID: PID0a74a Bittium HSAT Approved

Organization	Sleep sales center	Created	19.10.2021 11:40	Last data transfer	19.10.2021 11:41
Status	● Closed	Started	18.10.2021 23:30	Last update	19.10.2021 12:12
Assignment	Sales diag group 1 ▾	Ready for analysis	No date	Duration	1 / 1 day(s)

Recording data
Patient details
Anamneses
Patient diary
Recording details
Attachments

Cancel
Save

Patient details

Patient ID	Pat123456	Age (years)	55
	Patient ID must be at least 4 characters long and lower/uppercase letters, numbers and underscores are allowed. If Patient ID is left blank the system will generate it (for example PID0000A).	Height (cm)	175
		Weight (kg)	80
		BMI	26.12
		Gender	Female ▾

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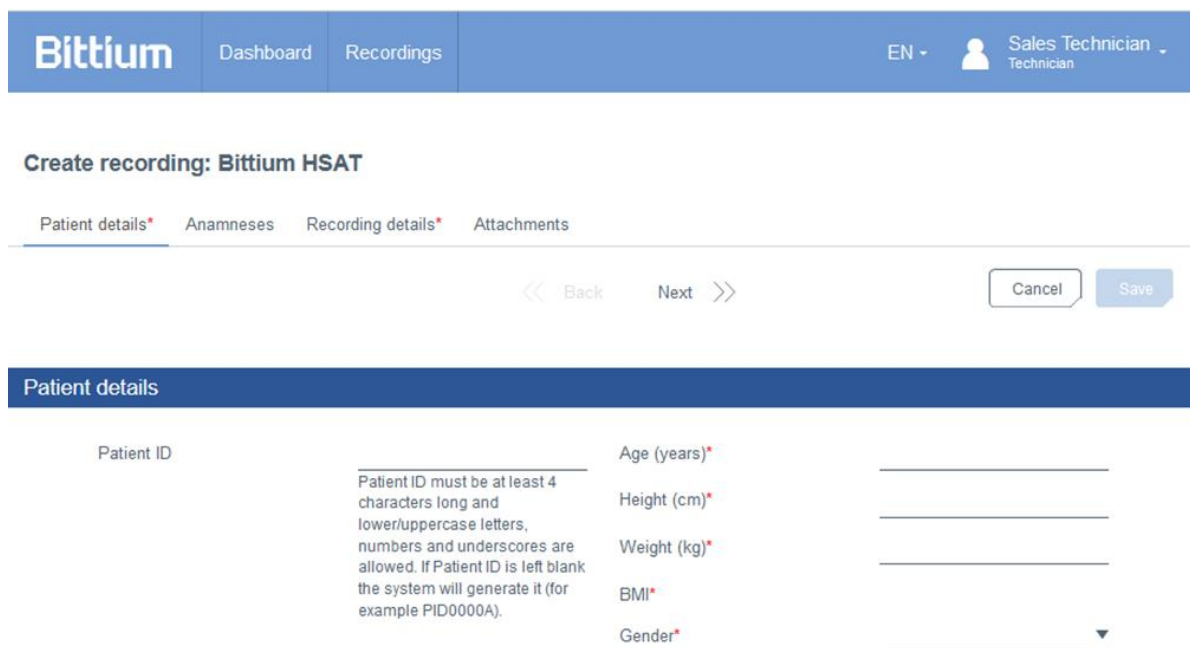
A screenshot of the Bittium MedicalSuite Center web interface. The top navigation bar includes the Bittium logo, 'Dashboard', 'Recordings', and user information for 'Sales Technician'. The main content area is titled 'Create recording: Bittium HSAT' and has tabs for 'Patient details*', 'Anamneses', 'Recording details*', and 'Attachments'. Below the tabs are navigation arrows for 'Back' and 'Next', and 'Cancel' and 'Save' buttons. The 'Patient details' section is highlighted in blue and contains several input fields: 'Patient ID' (with a note: 'Patient ID must be at least 4 characters long and lower/uppercase letters, numbers and underscores are allowed. If Patient ID is left blank the system will generate it (for example PID0000A).'), 'Age (years)*', 'Height (cm)*', 'Weight (kg)*', 'BMI*', and 'Gender*'. Each field has a corresponding input line.

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Create recording: Bittium HSAT

Patient details Anamneses Recording details Attachments

<< Back Next >>

Cancel

Save

Medication

Medication

Medication during the recording

No Yes

Medical history

Close family members diagnosed or treated for a sleep disorder

No Yes

Hypertension No Yes

Diabetes No Yes

Asthma No Yes

Hypothyreosis No Yes

Stuffy nose No Yes

Enlarged tonsils No Yes

Depression No Yes

Life

Smoking No Yes

Professional driver No Yes

Driving license class _____

Illustration 24 Skapa registrering – Anamnesfliken (1/2)

Chance of dozing off or falling asleep in the following situations

Sitting and reading	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Watching TV	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting, inactive in a public place (e.g. a theatre or a meeting)	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
As a passenger in a car for an hour without a break	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Lying down to rest in the afternoon when circumstances permit	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting and talking to someone	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting quietly after a lunch without alcohol	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
In a car, while stopped for a few minutes in the traffic	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High

Sleep history

Morning headache	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Dry mouth in the morning	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Daytime fatigue or snoring	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Daytime napping	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Heartburn	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Loss of concentration	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Memory loss	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Cramping at night	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Difficulties falling asleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Breathing breaks while sleeping	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Pain during night	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Snoring or snorting during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Waking up during the night or restless sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
A strong urge to move your feet or an uncomfortable sensation in your feet related to this urge during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Urinary urgency during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
If yes, how many times per night?	<input type="radio"/> Once	<input type="radio"/> Twice	<input type="radio"/> More often		
Average amount of sleep	_____ Hours				

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Create recording: Bittium HSAT

Patient details* Anamneses Recording details Attachments

<< Back Next >>

Cancel

Save

Recording details

Organization	ServiceProvider	Referring physician
Requested report language	English	
Reason for recording	<input type="text"/>	

Device information

Recorder S/N (21) 19 XXXXXXX

See the side of the device. Please add numbers after "19".

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Create recording: Bittium HSAT

Patient details Anamneses Recording details* Attachments

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Patient ID: **PID0a74a** Bittium HSAT

Ready for analysis |

Organization	Sleep test center	Created	29.12.2021 12:06	Last data transfer	29.12.2021 12:20
Status	● Active	Started	09.09.2021 23:30	Last update	29.12.2021 12:21
Assignment	None	Ready for analysis	No date	Duration	2 / 2 day(s)

[Recording data](#)
[Patient details](#)
[Anamneses](#)
[Patient diary](#)
[Recording details](#)
[Attachments](#)

Patient diary - 09.09.2021, 23:30 (06h 59m)

Please, ask the patient to answer the following questions about the recording.

When did you go to bed?	<input type="text" value="hh:mm"/>	In your estimation, when did you fall asleep?	<input type="text" value="hh:mm"/>
What time did you wake up in the morning?	<input type="text" value="hh:mm"/>	In your estimation, how many hours did you sleep?	<input type="text" value="hh:mm"/>
How did you sleep?	<input type="text"/>	Did you get out of bed during the recording?	<input type="text"/>
How did you sleep compared to usual?	<input type="text"/>		

Other comments

If the patient woke up during the recording, please write down the events.

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The screenshot displays the Bittium registration interface. At the top, a metadata table shows details for the recording:

Organization	Clinic A	Created	01.04.2019 15:34	Last data transfer	13.04.2019 12:00
Status	Preparing	Started	10.04.2019 12:00	Last update	13.04.2019 12:00
Assignment	None	Ended	No date	Duration	2 days

Below this, a navigation bar includes 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' section is visible with a checkbox for 'Include all attachments' and 'Cancel'/'Save' buttons. The main 'Recording data' section features a table with columns for Day no., Start time, Duration, Status, Quality, and Notes. Two entries are shown, with the second entry (Day 2/2) highlighted by a red box. An 'UPLOADS' window is also visible, showing a progress bar for Patient ID: 123455677888, with two files listed as 'Uploaded' and one as 'Preparing data - 2 / 2'.

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The screenshot shows the Bittium MedicalSuite interface. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is identified as 'Sales Technician' in 'EN'. The patient ID is 'PID0a74a' and the recording is for 'Bittium HSAT'. A 'Ready for analysis' button is visible. Below this, a table provides details about the recording:

Organization	Sleep test center	Created	29.12.2021 11:22	Last data transfer	29.12.2021 11:32
Status	Active	Started	16.12.2021 02:44	Last update	29.12.2021 11:33
Assignment	None	Ready for analysis	No date	Duration	2 / 2 day(s)

Below the table are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' button is present, along with a checkbox for 'Include all attachments' and 'Cancel'/'Save' buttons.

The 'Recording data' section shows a table with the following data:

Day No.	Start date of file	Duration	Status	Quality	Notes
1/1	16.12.2021 00:44	08h 10m	Open	Good	Add notes

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The screenshot shows the 'Recording data' section of the Bittium interface. At the top, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. Below the tabs, there is a 'Download' button and a checkbox for 'Include all attachments'. A 'Cancel' button and a 'Save' button are also present. The main content is a table with the following columns: Day No., Start date of file, Duration, Status, Quality, and Notes. The table contains three rows of data. The second row has a status of 'Open' and a quality of 'Good'. A dropdown menu is open for the 'Open' status, showing options: Pending, Done, Unusable, and Unusable. A tooltip for 'Unusable' is also visible, listing reasons: Corrupted recording data, Not enough analyzable data due to noisy signal or unattached recorder, Not enough sleep to write a statement, and Other (trs).

Day No.	Start date of file	Duration	Status	Quality	Notes
1/3	14.12.2021 23:42	04h 30m	Done	Fair	Add notes
2/3	14.12.2021 23:42	05h 00m	Open	Good	Add notes
3/3	14.12.2021 23:42	00h 15m	Unusable	Poor	Marked as "Unusable" <ul style="list-style-type: none">Corrupted recording dataNot enough analyzable data due to noisy signal or unattached recorderNot enough sleep to write a statementOther (trs)

Illustration 35 Välja registreringsdatastatus

Unusable recording data

Patient ID: [patient_id], 10/04/2019, 22:00 (08h 20m)

Why the recording data of this night is unusable?

- Corrupted recording data
- Not enough analyzable data due to noisy signal or unattached recorder
- Not enough sleep to write the statement
- Other

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Recording data Patient details Anamneses Recording details Attachments

Download

Cancel

Save

Reports

0 file(s)

Report can be added after the recording is closed.

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
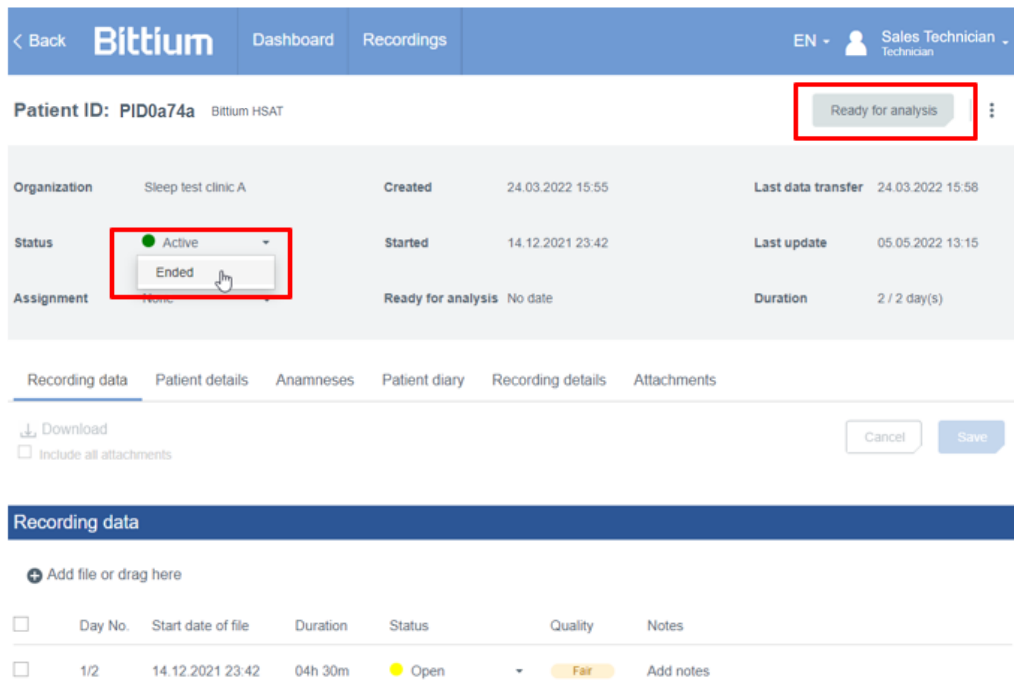
The screenshot shows the Bittium interface for a recording. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is identified as 'Sales Technician'. The patient ID is 'PID0a74a'. A 'Ready for analysis' button is highlighted with a red box. Below this, there is a table with recording details. The 'Status' dropdown menu is highlighted with a red box, showing 'Active' and 'Ended' options. The recording data table below shows a single entry with a status of 'Open' and a quality of 'Fair'.

Organization	Sleep test clinic A	Created	24.03.2022 15:55	Last data transfer	24.03.2022 15:58
Status	Active	Started	14.12.2021 23:42	Last update	05.05.2022 13:15
Assignment	None	Ready for analysis	No date	Duration	2 / 2 day(s)

Day No.	Start date of file	Duration	Status	Quality	Notes
1/2	14.12.2021 23:42	04h 30m	Open	Fair	Add notes

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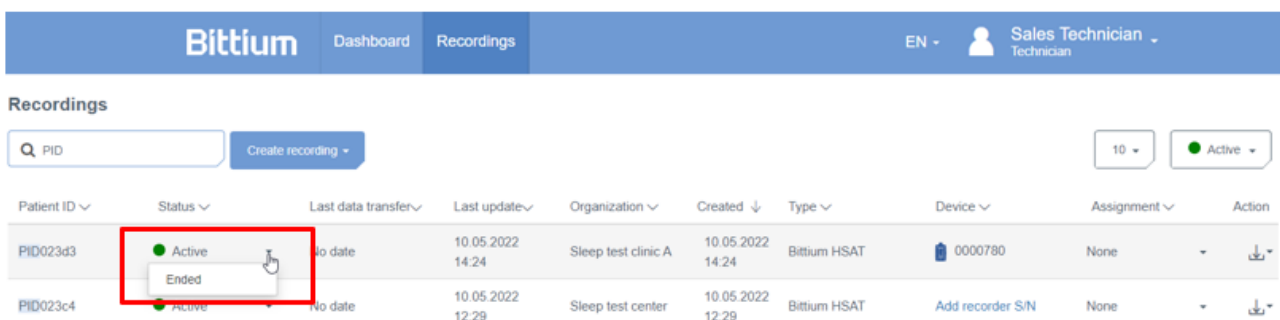The screenshot shows the Bittium interface for a list of recordings. The navigation bar is the same as in the previous screenshot. The 'Recordings' section has a search bar and a 'Create recording' button. A table lists recordings with columns for Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. The 'Status' dropdown menu for the first recording (PID023d3) is highlighted with a red box, showing 'Active' and 'Ended' options.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
PID023d3	Active	No date	10.05.2022 14:24	Sleep test clinic A	10.05.2022 14:24	Bittium HSAT	0000780	None	Download
PID023c4	Active	No date	10.05.2022 12:29	Sleep test center	10.05.2022 12:29	Bittium HSAT	Add recorder S/N	None	Download

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Ready for analysis

Patient ID **PID0a870_013696**

Are you sure this recording is ready for analysis?

- Please make sure the patient diary is filled or it is attached to "Other documents" section
- You will no longer be able to upload new recording data to it.

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Diagnostic report needed

All	Recording	Assignment	Ready for analysis	Duration	
	● PID0a74a	Test diag group 1	24.11.2021 09:10	1 day(s)	Analyze

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Organization DemoClinic1 Created 30.05.2023 13:56 Last data transfer 30.05.2023 14:01

Status ● Active Started 23.11.2021 23:42 Last update 25.01.2024 14:08

Assignment HSAT Analysis Ready for analysis No date Duration 1 / 1 day(s)

Groups

- Cardiscope Group
- Diagnostic group
- Diagnostic group (autoreports)
- HRV group (autoreports)
- ✓ HSAT Analysis

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Diagnostic report needed

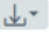
All Not assigned	Recording	Assignment	Ready for analysis	Duration	
● PatientZero		Test diag group 1 ▾	24.11.2021 09:10	1 day(s)	Analyze 

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1 of 3 Automatic Zoom

Patient

Patient ID:	23-30-48	Height:	185cm	Weight:	85kg
BMI:	24.8	Age:	58 years	Gender:	Male

Summary

Apnea-hypopnea index: 8.7/h
(Supine: 15.4/h, Non-supine: 2.1/h)

Oxygen Desaturation Index (ODI3): 8.7/h

AHI

Normal Mild Moderate Severe

5 15 30

ODI3

Recording date	15/10/2021	Recording: TRT	08h 36m (23:30 - 08:07)
Evaluation period	08h 16m (23:43 - 08:00)		

Apnea-hypopnea index (AHI) displayed here is the total number of respiratory events (Apneas + Hypopneas) divided by Evaluation period. Evaluation period was determined based on measured signals and patient diary

Statement

Findings

Nearly all occurrences of respiratory events are in the supine position. The recording was technically successful and of high quality.

Conclusion

The recording findings are consistent with mild obstructive sleep apnea. Avoiding sleeping in a supine position (positional therapy), for example by placing a tennis ball in the back of the pajamas or wearing a sleep apnea belt, might be sufficient treatment. It is recommended to confirm the treatment outcome with a new recording.

23/02/2022

Dr. Sleep Apnea

This report is the consulting doctor's opinion based on the monitoring and the available background information. The final conclusions are always made by the patient's attending doctor who has the best

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Accept the report

Once you accept the report, the recording will be closed. Are you sure you want to accept the report?

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< Back **Bittium** Dashboard Recordings EN Tiego TechnicianUser Technician

Patient ID: PID0a743_013381 Bittium HSAT Close

Organization	Sleep test center	Created	13.12.2021 09:29	Last data transfer	13.12.2021 09:31
Status	● Ended ▾ Closed	Started	24.06.2021 00:00	Last update	29.12.2021 16:20
Assignment	Daniela DiagnosticUser ▾	Ready for analysis	13.12.2021 09:44	Duration	1 / 1 day(s)

Recording data Patient details Anamneses Patient diary Recording details Attachments

Download Cancel Save

Include all attachments

Illustration 47 Stänga registreringen i vyn Enskild registrering

Bittium Dashboard Recordings EN Sales Technician Technician

Recordings

Create recording ▾ 10 ▾ All ▾

Patient ID ▾	Status ▾	Last data transfer ▾	Last update ▾	Organization ▾	Created ▾	Type ▾	Device ▾	Assignment ▾	Action
PID0a743	● Ended ▾ Closed	13.12.2021 09:31	29.12.2021 16:20	Sleep test center	13.12.2021 09:29	Bittium HSAT	0000552	Daniela DiagnosticUser ▾	

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Close recording

If you want to upload a report, click "Add file" to select a report or drag-and-drop a file before closing the recording.

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Latest reports				
Recording	Organization	Added	Approval	
● PID0a74a	DemoClinic2	07.10.2020 17:49	07.10.2020 17:54	 

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Recordings									
Q PID0a74a		Create recording -		10 -		All -			
Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
PID0a74a	● Closed	14.12.2021 15:47	17.12.2021 15:22	Sleep test center	14.12.2021 15:47	Bittium HSAT	0123456	None	 

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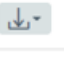
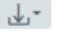
Latest reports				
Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	 
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	Approve	 

Illustration 53 Godkänn-knapp på "Senaste rapporter"-kortet på instrumentpanelen

The screenshot shows the Bittium interface for a patient with ID PID0a74a. The top navigation bar includes 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is identified as 'Sales Technician'. The patient's name is 'Bittium HSAT'. A red box highlights the 'Approve' button in the top right corner. Below this, a table displays patient details:

Organization	Sleep test center	Created	14.12.2021 15:47	Last data transfer	14.12.2021 15:47
Status	● Closed	Started	24.02.2021 00:00	Last update	17.12.2021 15:22
Assignment	None	Ready for analysis	14.12.2021 15:49	Duration	1 / 1 day(s)

Below the table are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. At the bottom, there are options to 'Download' (with a checkbox for 'Include all attachments') and buttons for 'Cancel' and 'Save'.

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The confirmation dialog box has a blue header with the title 'Confirmation'. The main text asks: 'Are you sure you want to approve the reports of Patient ID: 01-10-56? After approval, reports will be visible for the clinic.' At the bottom, there are two buttons: 'Cancel' and 'Approve'.

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[← Back](#)
Bittium
Dashboard
Recordings
EN ▼

Sales Technician
Technician

Patient ID: **PID0a74a** Bittium HSAT Approved

Organization	Sleep test center	Created	14.12.2021 15:47	Last data transfer	14.12.2021 15:47
Status	● Closed	Started	24.02.2021 00:00	Last update	31.12.2021 10:13
Assignment	None ▼	Ready for analysis	14.12.2021 15:49	Duration	1 / 1 day(s)

[Recording data](#)
[Patient details](#)
[Anamneses](#)
[Patient diary](#)
[Recording details](#)
[Attachments](#)

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Latest reports

Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 <small>Not assigned</small>	16.12.2021 09:48	
● PID0a74a	Sleep test center	15.12.2021 15:16 <small>Not assigned</small>	16.12.2021 09:48	

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Latest reports

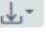
Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	 
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	16.12.2021 09:48	 

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3 file(s)

+ Add file or drag here

<input type="checkbox"/>	Filename	Added			
<input type="checkbox"/>	recording-information.pdf				
<input type="checkbox"/>	Document_1.txt	19.12.2021 14:53			

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Patient ID: PID0a870_013696 Bittium HSAT

Organization	Sleep test center	Created	21.12.2021 13:38	Last data transfer	21.12.2021 13:39
Status	● Ended	Started	23.05.2021 22:53	Last update	23.12.2021 13:41
Assignment	None	Ready for analysis	21.12.2021 13:42	Duration	1 / 1 day(s)

Recording data Patient details Anamneses Patient diary Recording details Attachments

Download
 Include all attachments

Cancel Save

Recording data

<input type="checkbox"/>	Day No.	Start date of file	Duration	Status	Quality	Notes
<input type="checkbox"/>	1/1	23.05.2021 19:53	08h 00m	● Open	● Poor	Add notes

Illustration 62 Avvisa registrering i vyn Enskild registrering

Patient ID: PID020c5_005914 Bittium HSAT

Organization	Evismo test center	Created	07.09.2021 15:04	Last data transfer	No date
Status	○ Rejected	Started	No date	Last update	22.09.2021 14:37
Assignment	None	Ready for analysis	No date	Duration	

Recording data Patient details Anamneses Patient diary Recording details Attachments

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Reject recording

Patient ID: **PID023d3**

The recorder 0000780 is currently in use in the selected recording. It will be released when the recording is rejected. Are you sure you want to reject the recording?

This action cannot be undone.

Why do you want to reject the recording?

- Corrupted recording data
- Not enough analyzable data due to noisy signal or unattached recorder
- Not enough sleep to write a statement
- Other

Cancel

Reject

Delete recording

Patient ID: **PID023d3**

The recorder 0000780 is currently in use in the selected recording. It will be released when the recording is deleted. Are you sure you want to delete the recording?

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Illustration 66 Inspektera avvisade registreringar i registreringsvyn

Patient ID	Test	Bittium HSAT			
Organization		Created	20.12.2021 14:06	Last data transfer	20.12.2021 14:58
Status	Rejected	Started	20.12.2021 14:09	Last update	21.12.2021 15:11

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Navigation: < Back | Bittium | Dashboard | Recordings | EN - | Sales Technician | Technician

Patient ID: PID0a74a | Bittium HSAT | Approve

Organization	Sleep test center	Created	16.12.2021 15:27	Last data transfer	16.12.2021 16:24
Status	● Closed	Started	16.12.2021 15:36	Last update	21.12.2021 13:58
Assignment	None	Ready for analysis	17.12.2021 14:24	Duration	2 / 2 day(s)

Recording data | Patient details | Anamneses | Patient diary | Recording details | Attachments

Download
 Include all attachments

Cancel Save

Recording data

<input type="checkbox"/>	Day No.	Start date of file	Duration	Status	Quality	Notes	
<input type="checkbox"/>	1/2	16.12.2021 15:36	00h 29m	● Open	Poor	Add notes	📄
<input checked="" type="checkbox"/>	2/2	17.12.2021 10:01	03h 00m	● Open	Poor	Add notes	📄

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Copyright 2024 Bittium Biosignals Ltd. Med ensamrätt. Informationen i detta dokument kan ändras utan föregående meddelande. Bittium Biosignals Ltd behåller äganderätten till och alla andra rättigheter till det material som uttrycks i detta dokument. Ingen del av innehållet i detta dokument får kopieras utan föregående skriftligt tillstånd från Bittium Biosignals Ltd.
Bittium Biosignals Ltd