Bittium

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
The screenshot shows the Bittium MedicalSuite Center dashboard. At the top, there is a navigation bar with 'Bittium', 'Dashboard', and 'Recordings' tabs. The user is logged in as 'Sales Technician' in English. The dashboard features a 'Create recording' button and an 'Edit dashboard' button. Below these are four summary cards: 'Created' (0), 'Active' (7), 'Ended' (0), and 'Closed' (6). The 'Active recordings' section displays a table with columns for Recording ID, Started, Type, and Duration. The 'Latest reports' section shows a table with columns for Recording ID, Organization, Updated, and Approval, along with icons for print and download.

Recording	Started	Type	Duration
PID00007	-	Bittium HSAT	0 / 0 day(s)
KK02	26.10.2021 23:20	Bittium HSAT	1 / 1 day(s)
PID00009	16.10.2021 00:09	Bittium HSAT	1 / 1 day(s)
PID00008	15.10.2021 23:30	Bittium HSAT	1 / 1 day(s)
23_18_02	09.08.2021 23:18	Bittium HSAT	1 / 1 day(s)
23_48_42	02.08.2021 23:48	Bittium HSAT	1 / 1 day(s)
22_47_08	20.04.2021 22:47	Bittium HSAT	1 / 1 day(s)

Recording	Organization	Updated	Approval
Pat123456	Sleep sales center	19.10.2021 12:12 Sales diag group 1	19.10.2021 12:12
PID0000b	Sleep sales center	18.10.2021 18:15 Sales diag group 1	18.10.2021 18:16

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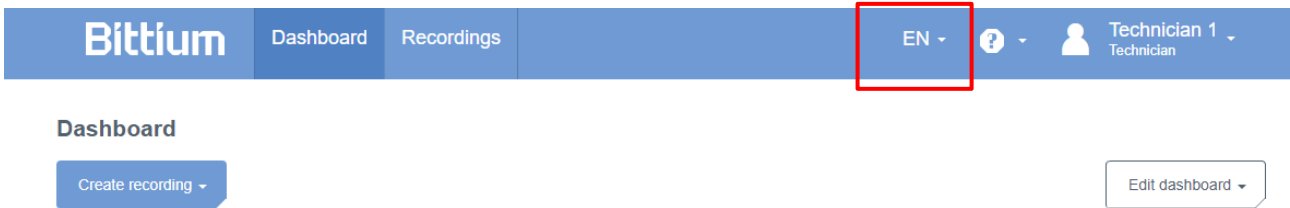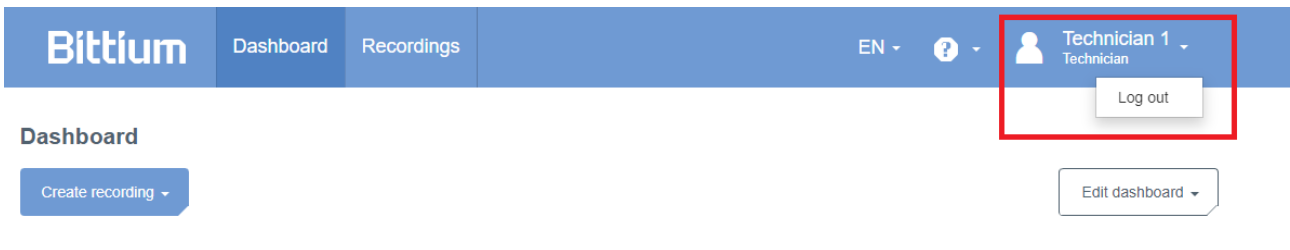
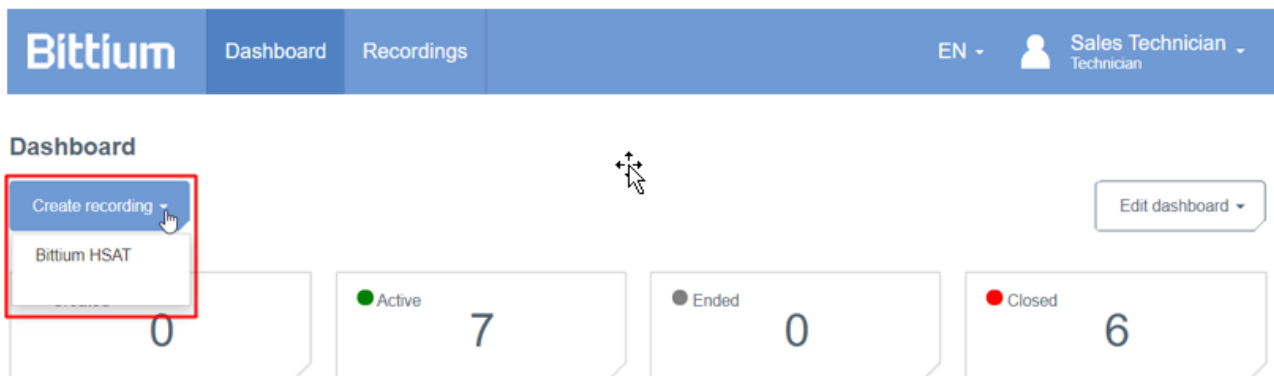
Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
KK02	Active	28.10.2021 15:25	28.10.2021 15:46	Sleep sales center	28.10.2021 15:23	Bittium HSAT	0000568	Sales diag group 1	Download
Pat123456	Closed	19.10.2021 11:41	19.10.2021 12:12	Sleep sales center	19.10.2021 11:40	Bittium HSAT	0000673	Sales diag group 1	Download
PID0000b	Closed	18.10.2021 17:48	18.10.2021 18:16	Sleep sales center	18.10.2021 17:47	Bittium HSAT	0000673	Sales diag group 1	Download
PID0000a	Closed	18.10.2021 16:19	18.10.2021 18:15	Sleep sales center	18.10.2021 16:18	Bittium HSAT	0000536	Sales diag group 1	Download
PID00009	Active	16.10.2021 08:30	09.11.2021 10:16	Sleep sales center	16.10.2021 08:29	Bittium HSAT	0000536	Sales diag group 1	Download
PID00008	Active	16.10.2021 08:25	16.10.2021 09:04	Sleep sales center	16.10.2021 08:25	Bittium HSAT	0000568	Sales diag group 1	Download
Test008	Closed	15.10.2021 19:01	18.10.2021 15:26	Sleep sales center	15.10.2021 19:00	Bittium HSAT	0000673	Sales diag group 1	Download
TEST007	Closed	15.10.2021 18:46	15.10.2021 19:14	Sleep sales center	15.10.2021 18:43	Bittium HSAT	0000673	Sales diag group 1	Download
PID00007	Active	No date	14.10.2021 17:22	Sleep sales center	14.10.2021 17:18	Bittium HSAT	1234567	Sales diag group 1	Download
23_45_08	Closed	01.10.2021 16:17	14.10.2021 17:19	Sleep sales center	01.10.2021 16:16	Bittium HSAT	0123456	Sales diag group 1	Download

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
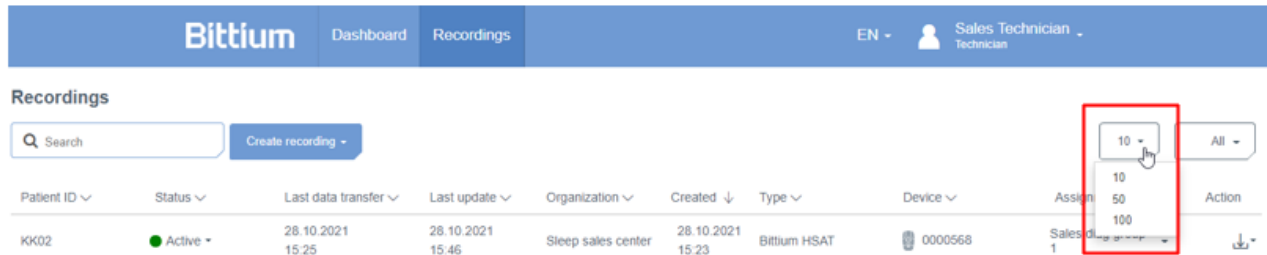
The screenshot shows the Bittium Recordings interface. At the top, there is a navigation bar with 'Bittium', 'Dashboard', and 'Recordings' tabs. The user is logged in as 'Sales Technician'. Below the navigation bar, there is a search bar and a 'Create recording' button. A table of recordings is displayed with columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open over the table, showing options for the number of records to display: 10, 50, and 100. The '10' option is selected and highlighted with a red box.

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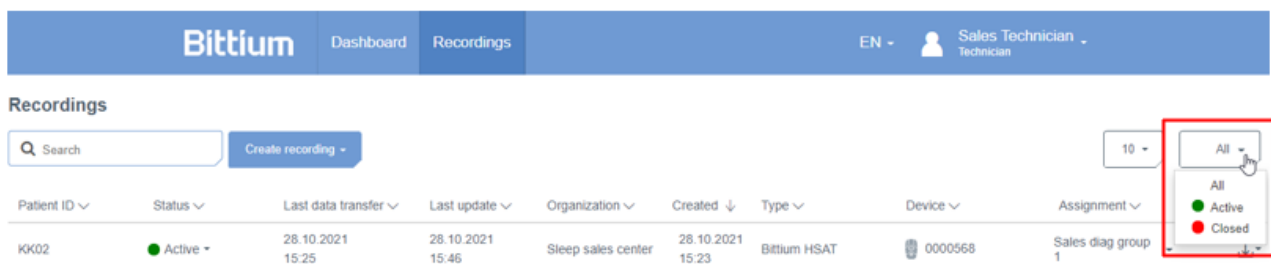
The screenshot shows the Bittium Recordings interface. At the top, there is a navigation bar with 'Bittium', 'Dashboard', and 'Recordings' tabs. The user is logged in as 'Sales Technician'. Below the navigation bar, there is a search bar and a 'Create recording' button. A table of recordings is displayed with columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open over the table, showing options for the status of records to display: All, Active, and Closed. The 'All' option is selected and highlighted with a red box.

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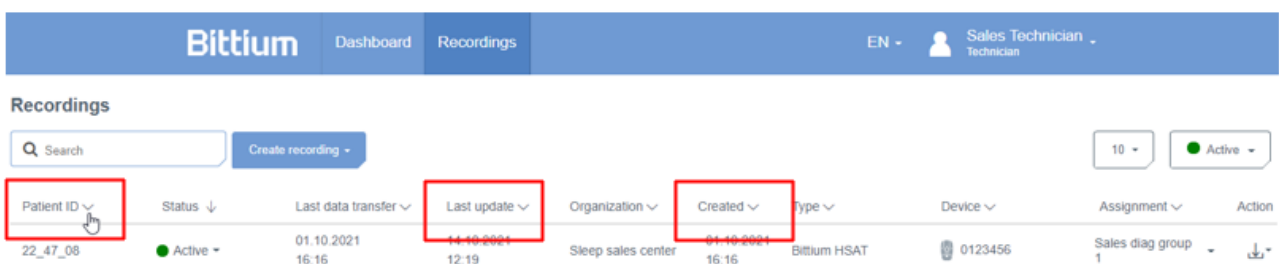The screenshot shows the Bittium Recordings interface. At the top, there is a navigation bar with 'Bittium', 'Dashboard', and 'Recordings' tabs. The user is logged in as 'Sales Technician'. Below the navigation bar, there is a search bar and a 'Create recording' button. A table of recordings is displayed with columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open over the table, showing options for the sorting order of records: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. The 'Patient ID' option is selected and highlighted with a red box.

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The screenshot shows the Bittium Recordings interface. At the top, there are navigation tabs for 'Dashboard' and 'Recordings', and a user profile for 'Technician 1'. Below the navigation is a search bar and a 'Create recording' button. The main area contains a table with columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open for the 'Assignment' column of the first row, showing options like 'Cardioscope Group', 'Diagnostic group', and 'HSAT Analysis'. The 'HSAT Analysis' option is selected, and a sub-menu is visible showing 'Users in HSAT Analysis' with 'Dr. Hanna HSAT' listed.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
testPDF	Closed	23.01.2024 15:14	25.01.2024 13:41	ServiceProvider	23.01.2024 15:14	Bittium	0000533	HSAT Analysis	Download
HSAT_ECG	Ended	16.01.2024 16:13	16.01.2024 16:17	DemoClinic1	16.01.2024 16:13	Bittium			Download
HSAT_2_RIP_Belts	Active	16.01.2024 16:11	25.01.2024 10:10	DemoClinic1	16.01.2024 16:11	Bittium			Download
PID00314	Ended	14.12.2023 15:06	19.12.2023 13:09	DemoClinic1	14.12.2023 15:05	Bittium			Download
PID002f3	Closed	22.11.2023 11:37	22.11.2023 11:41	DemoClinic1	22.11.2023 11:37	Bittium			Download

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The screenshot shows the Bittium Recordings interface. At the top, there are navigation tabs for 'Dashboard' and 'Recordings', and a user profile for 'Sales Technician'. Below the navigation is a search bar and a 'Create recording' button. The main area contains a table with columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open for the 'Action' column of the first row, showing options like 'Recording data', 'Recording data 1 day(s)', 'Reports', and 'Other documents'. The 'Recording data' option is selected, and a sub-menu is visible showing '6169a6099ec5e8000acd5a02_statement.pdf' and 'recording-information.pdf'.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
Test008	Closed	15.10.2021 19:01	18.10.2021 15:26	Sleep sales center	15.10.2021 19:00	Bittium HSAT	0000533	Sales diag group 1	Download
TEST007	Closed	15.10.2021 18:46	15.10.2021 19:14	Sleep sales center	15.10.2021 18:43	Bittium HSAT	0000533		Download
PID0000b	Closed	18.10.2021 17:48	18.10.2021 18:15	Sleep sales center	18.10.2021 17:47	Bittium HSAT	0000533		Download
PID0000a	Closed	18.10.2021 16:19	18.10.2021 16:15	Sleep sales center	18.10.2021 16:18	Bittium HSAT	0000533		Download

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**Patient ID: Pat123456** Bittium HSAT

Approved

Organization	Sleep sales center	Created	19.10.2021 11:40	Last data transfer	19.10.2021 11:41
Status	● Closed	Started	18.10.2021 23:30	Last update	19.10.2021 12:12
Assignment	Sales diag group 1	Ready for analysis	No date	Duration	1 / 1 day(s)

[Recording data](#)[Patient details](#)[Anamneses](#)[Patient diary](#)[Recording details](#)[Attachments](#)

Cancel

Save

Patient details

Patient ID	Pat123456	Age (years)	55
	<small>Patient ID must be at least 4 characters long and lower/uppercase letters, numbers and underscores are allowed. If Patient ID is left blank the system will generate it (for example PID0000A).</small>	Height (cm)	175
		Weight (kg)	80
		BMI	26.12
		Gender	Female

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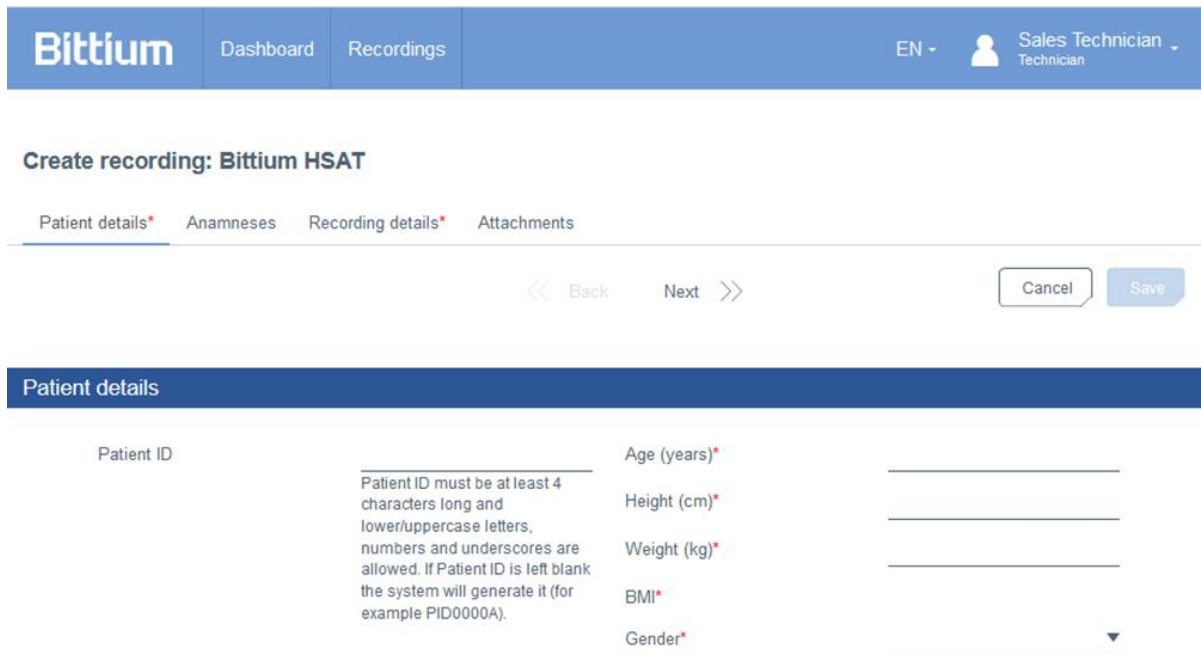
A screenshot of the 'Create recording: Bittium HSAT' form. The form has a blue header with the Bittium logo and navigation tabs: 'Dashboard', 'Recordings', 'EN', and 'Sales Technician Technician'. Below the header, the title 'Create recording: Bittium HSAT' is displayed. There are four tabs: 'Patient details*', 'Anamneses', 'Recording details*', and 'Attachments'. The 'Patient details*' tab is active. Below the tabs, there are navigation arrows: '<< Back', 'Next >>', 'Cancel', and 'Save'. The 'Patient details' section contains several input fields: 'Patient ID' with a note 'Patient ID must be at least 4 characters long and lower/uppercase letters, numbers and underscores are allowed. If Patient ID is left blank the system will generate it (for example PID0000A).', 'Age (years)*', 'Height (cm)*', 'Weight (kg)*', 'BMI*', and 'Gender*'. Each field has a corresponding input line.

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Bittium Dashboard Recordings EN - Sales Technician Technician

Create recording: Bittium HSAT

Patient details Anamneses Recording details Attachments

<< Back Next >> Cancel Save

Medication

Medication

Medication during the recording No Yes

Medical history

Close family members diagnosed or treated for a sleep disorder No Yes

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Chance of dozing off or falling asleep in the following situations

Sitting and reading	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Watching TV	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting, inactive in a public place (e.g. a theatre or a meeting)	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
As a passenger in a car for an hour without a break	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Lying down to rest in the afternoon when circumstances permit	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting and talking to someone	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting quietly after a lunch without alcohol	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
In a car, while stopped for a few minutes in the traffic	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High

Sleep history

Morning headache	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Dry mouth in the morning	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Daytime fatigue or snoring	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Daytime napping	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Heartburn	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Loss of concentration	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Memory loss	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Cramping at night	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Difficulties falling asleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Breathing breaks while sleeping	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Pain during night	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Snoring or snorting during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Waking up during the night or restless sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
A strong urge to move your feet or an uncomfortable sensation in your feet related to this urge during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Urinary urgency during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
If yes, how many times per night?	<input type="radio"/> Once	<input type="radio"/> Twice	<input type="radio"/> More often		
Average amount of sleep	_____ Hours				

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Create recording: Bittium HSAT

Patient details* Anamneses **Recording details** Attachments

<< Back Next >>

Cancel

Save


Recording details

Organization	ServiceProvider	Referring physician
Requested report language	English	
Reason for recording		

Device information

Recorder S/N (21) 19 XXXXXXX

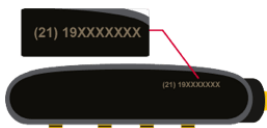See the side of the device. **Please add numbers after "19".**

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Create recording: Bittium HSAT

Patient details Anamneses Recording details* Attachments

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Patient ID: **PID0a74a** Bittium HSAT

Ready for analysis | ⋮

Organization	Sleep test center	Created	29.12.2021 12:06	Last data transfer	29.12.2021 12:20
Status	● Active	Started	09.09.2021 23:30	Last update	29.12.2021 12:21
Assignment	None	Ready for analysis	No date	Duration	2 / 2 day(s)

Recording data Patient details Anamneses **Patient diary** Recording details Attachments

Cancel

Save

Patient diary - 09.09.2021, 23:30 (06h 59m)

Please, ask the patient to answer the following questions about the recording.

When did you go to bed?	hh:mm	In your estimation, when did you fall asleep?	hh:mm
What time did you wake up in the morning?	hh:mm	In your estimation, how many hours did you sleep?	hh:mm
How did you sleep?		Did you get out of bed during the recording?	
How did you sleep compared to usual?			

Other comments

If the patient woke up during the recording, please write down the events.

 Add new event

Event			
Time	00:51	Duration	hh:mm
Event	Recorder button pressed		

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
Status	Uploading	Created	01.04.2019 15:34	Last update	01.04.2019 15:34
Assignment	Mabel Hicks	Started	---	Duration	---

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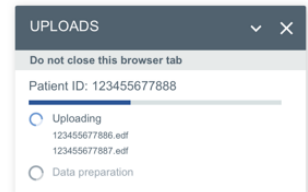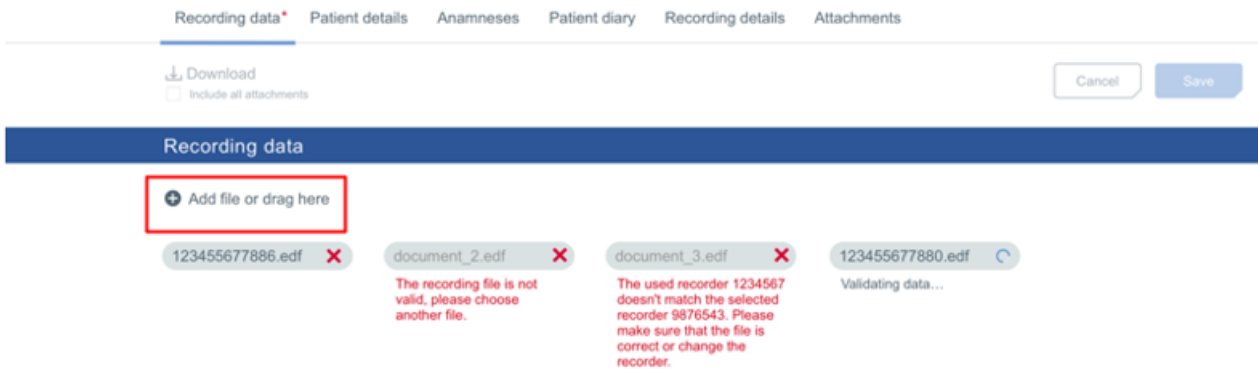
Organization Clinic A Created 01.04.2019 15:34 Last data transfer 13.04.2019 12:00

Status **Preparing** Started 10.04.2019 12:00 Last update 13.04.2019 12:00

Assignment None Ended No date Duration 2 days

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Patient ID: PID0a74a Bittium HSAT

Ready for analysis

Organization	Sleep test center	Created	29.12.2021 11:22	Last data transfer	29.12.2021 11:32
Status	● Active	Started	16.12.2021 02:44	Last update	29.12.2021 11:33
Assignment	None	Ready for analysis	No date	Duration	2 / 2 day(s)

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Recording data

+ Add file or drag here

<input type="checkbox"/>	Day No.	Start date of file	Duration	Status	Quality	Notes
<input type="checkbox"/>	1/1	16.12.2021 00:44	08h 10m	● Open	● Good	Add notes

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The screenshot shows the 'Recording data' section of the Bittium software. At the top, there are navigation tabs: 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. Below the tabs, there are options for 'Download' and 'Include all attachments', along with 'Cancel' and 'Save' buttons. The main area displays a table of recording data with columns for 'Day No.', 'Start date of file', 'Duration', 'Status', 'Quality', and 'Notes'. Three rows are visible, each with a checkbox on the left. The 'Status' column for the second row is highlighted with a red box, and a dropdown menu is open, showing options: 'Open', 'Pending', 'Done', 'Unusable', and 'Unusable'. The 'Quality' column for the second row is 'Good', and for the third row, it is 'Poor'. A tooltip is visible for the 'Poor' quality, listing reasons for being marked as 'Unusable': 'Corrupted recording data', 'Not enough analyzable data due to noisy signal or unattached recorder', 'Not enough sleep to write a statement', and 'Other (trs)'.

Day No.	Start date of file	Duration	Status	Quality	Notes
1/3	14.12.2021 23:42	04h 30m	Done	Fair	Add notes
2/3	14.12.2021 23:42	05h 00m	Open	Good	Add notes
3/3	14.12.2021 23:42	00h 15m	Unusable	Poor	Marked as "Unusable" <ul style="list-style-type: none">Corrupted recording dataNot enough analyzable data due to noisy signal or unattached recorderNot enough sleep to write a statementOther (trs)

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Unusable recording data

Patient ID: [patient_id], 10/04/2019, 22:00 (08h 20m)

Why the recording data of this night is unusable?

Corrupted recording data

Not enough analyzable data due to noisy signal or unattached recorder

Not enough sleep to write the statement

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Recording data Patient details Anamneses Recording details Attachments

Download Cancel Save

Reports 0 file(s)

Report can be added after the recording is closed.

Other documents 1 file(s)

+ Add file or drag here

<input type="checkbox"/>	Filename	Added	
<input type="checkbox"/>	recording-information.pdf		  

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The screenshot shows the Bittium MedicalSuite interface for a patient recording. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is logged in as 'Sales Technician'.

The patient ID is 'PID0a74a' (Bittium HSAT). A red box highlights a 'Ready for analysis' button in the top right corner.

The recording details are as follows:

Field	Value
Organization	Sleep test clinic A
Created	24.03.2022 15:55
Last data transfer	24.03.2022 15:58
Status	Active (dropdown menu is open, showing 'Ended' as an option)
Started	14.12.2021 23:42
Last update	05.05.2022 13:15
Assignment	None
Ready for analysis	No date
Duration	2 / 2 day(s)


Below the details, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. The 'Recording data' tab is selected.

There is a 'Download' button and a checkbox for 'Include all attachments'. 'Cancel' and 'Save' buttons are also present.

The 'Recording data' section shows a table with the following data:

Day No.	Start date of file	Duration	Status	Quality	Notes
1/2	14.12.2021 23:42	04h 30m	Open	Fair	Add notes

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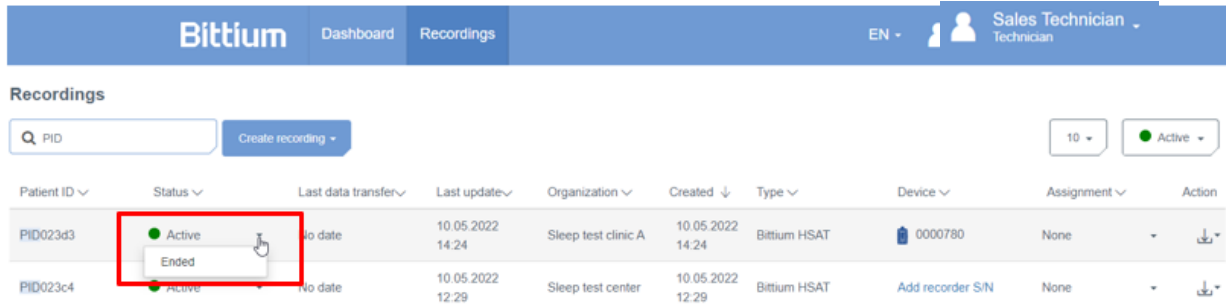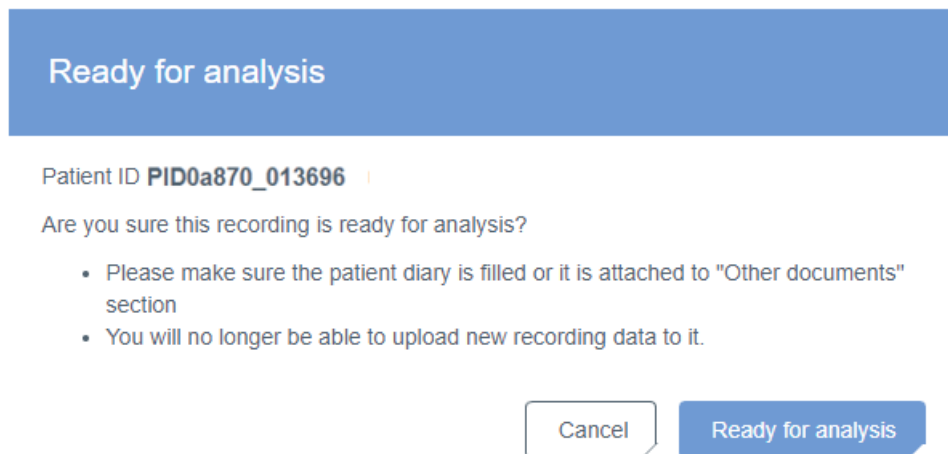
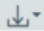
Diagnostic report needed

All Not assigned	Assignment	Ready for analysis	Duration	
Recording				
● PID0a74a	Test diag group 1 ▾	24.11.2021 09:10	1 day(s)	Analyze 

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The screenshot shows the Bittium software interface for a patient's HSAT analysis. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The patient ID is 'PID00298' and the device is 'Bittium HSAT'. A 'Ready for analysis' button is visible. Below this, a table displays patient information: Organization (DemoClinic1), Status (Active), Created (30.05.2023 13:56), Started (23.11.2021 23:42), Last data transfer (30.05.2023 14:01), Last update (25.01.2024 14:08), Ready for analysis (No date), and Duration (1 / 1 day(s)). A dropdown menu is open for 'Assignment', showing a search bar and a list of groups: Cardioscope Group, Diagnostic group, Diagnostic group (autoreports), HRV group (autoreports), and HSAT Analysis (selected). Below the groups, there is a section for 'Users in HSAT Analysis' with 'Dr. Hanna HSAT' listed. At the bottom right, there are 'Cancel' and 'Save' buttons.

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Diagnostic report needed

All Not assigned	Recording	Assignment	Ready for analysis	Duration	
● PatientZero		Test diag group 1	24.11.2021 09:10	1 day(s)	Analyze

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Summary and statement Analysis Report

Accept report

1 of 3 Automatic Zoom

Patient

Patient ID:	23-30-48	Height:	185cm	Weight:	85kg
BMI:	24.8	Age:	58 years	Gender:	Male

Summary

Apnea-hypopnea index: 8.7/h
(Supine: 15.4/h, Non-supine: 2.1/h)

Oxygen Desaturation Index (ODI3): 8.7/h

AHI scale: Normal (0-5), Mild (5-15), Moderate (15-30), Severe (30-38). ODI3 is marked at 8.7/h.

Recording date	15/10/2021	Recording: TRT	08h 36m (23:30 - 08:07)
Evaluation period	08h 16m (23:43 - 08:00)		

Apnea-hypopnea index (AHI) displayed here is the total number of respiratory events (Apneas + Hypopneas) divided by Evaluation period. Evaluation period was determined based on measured signals and patient diary

Statement

Findings
Nearly all occurrences of respiratory events are in the supine position. The recording was technically successful and of high quality.

Conclusion
The recording findings are consistent with mild obstructive sleep apnea. Avoiding sleeping in a supine position (positional therapy), for example by placing a tennis ball in the back of the pajamas or wearing a sleep apnea belt, might be sufficient treatment. It is recommended to confirm the treatment outcome with a new recording.

23/02/2022
Dr. Sleep Apnea
This report is the consulting doctor's opinion based on the monitoring and the available background information. The final conclusions are always made by the patient's attending doctor who has the best

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Accept the report

Once you accept the report, the recording will be closed. Are you sure you want to accept the report?

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Patient ID: PID0a743_013381 Bittium HSAT Close

Organization	Sleep test center	Created	13.12.2021 09:29	Last data transfer	13.12.2021 09:31
Status	● Ended Closed	Started	24.06.2021 00:00	Last update	29.12.2021 16:20
Assignment	Daniela DiagnosticUser	Ready for analysis	13.12.2021 09:44	Duration	1 / 1 day(s)

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Navigation: Bittium Dashboard Recordings EN - Sales Technician Technician

Recordings

Search: PID0a743 Create recording 10 All

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
PID0a743	● Ended Closed	13.12.2021 09:31	29.12.2021 16:20	Sleep test center	13.12.2021 09:29	Bittium HSAT	0000552	Daniela DiagnosticUser	

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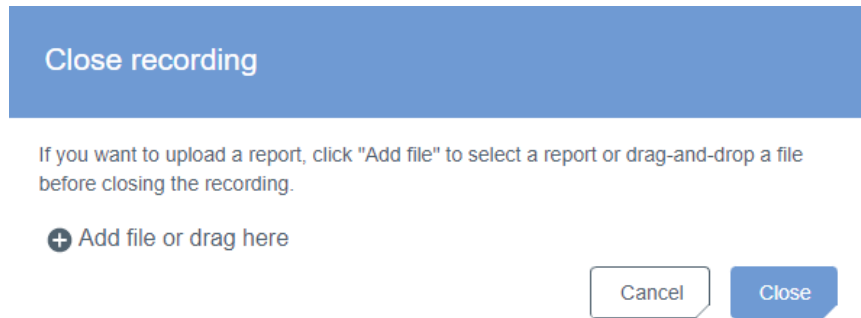
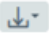
Latest reports				
Recording	Organization	Added	Approval	
● PID0a74a	DemoClinic2	07.10.2020 17:49	07.10.2020 17:54	 

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Recordings

Search: PID0a74a Create recording - 10 - All -

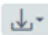
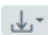
Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
PID0a74a	● Closed	14.12.2021 15:47	17.12.2021 15:22	Sleep test center	14.12.2021 15:47	Bittium HSAT	0123456	None	

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Latest reports				
Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	 
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	Approve	 

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< Back **Bittium** Dashboard Recordings EN - Sales Technician Technician

Patient ID: PID0a74a Bittium HSAT Approve

Organization	Sleep test center	Created	14.12.2021 15:47	Last data transfer	14.12.2021 15:47
Status	● Closed	Started	24.02.2021 00:00	Last update	17.12.2021 15:22
Assignment	None	Ready for analysis	14.12.2021 15:49	Duration	1 / 1 day(s)

Recording data Patient details Anamneses Patient diary Recording details Attachments

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Confirmation

Are you sure you want to approve the reports of Patient ID: 01-10-56? After approval, reports will be visible for the clinic.

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Patient ID: PID0a74a Bittium HSAT Approved

Organization	Sleep test center	Created	14.12.2021 15:47	Last data transfer	14.12.2021 15:47
Status	● Closed	Started	24.02.2021 00:00	Last update	31.12.2021 10:13
Assignment	None	Ready for analysis	14.12.2021 15:49	Duration	1 / 1 day(s)

[Recording data](#) [Patient details](#) [Anamneses](#) [Patient diary](#) [Recording details](#) [Attachments](#)

Download Include all attachments Cancel Save

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Latest reports

Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	16.12.2021 09:48	

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
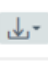
Latest reports

Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	 
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	16.12.2021 09:48	 

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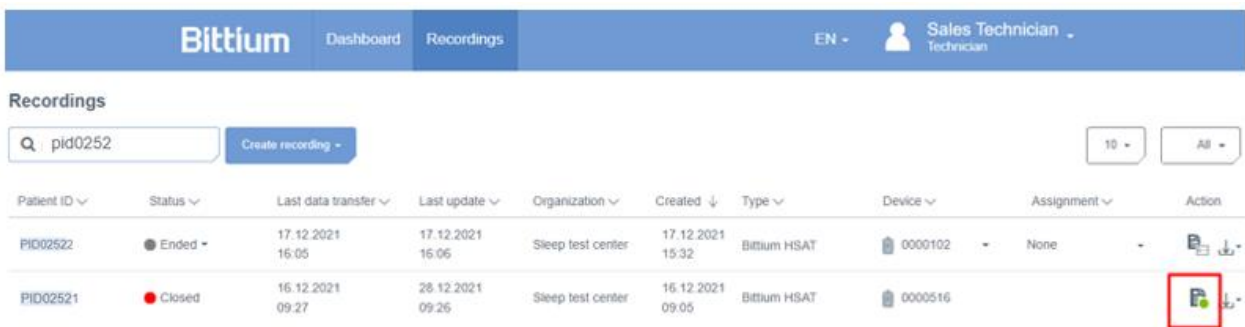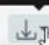
3 file(s)

+ Add file or drag here

<input type="checkbox"/>	Filename	Added			
<input type="checkbox"/>	recording-information.pdf				
<input type="checkbox"/>	Document_1.txt	19.12.2021 14:53			

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< Back **Bittium** Dashboard Recordings EN Sales Technician Technician

Patient ID: PID0a870_013696 Bittium HSAT

Organization	Sleep test center	Created	21.12.2021 13:38	Last data transfer	21.12.2021 13:39
Status	● Ended	Started	23.05.2021 22:53	Last update	23.12.2021 13:41
Assignment	None	Ready for analysis	21.12.2021 13:42	Duration	1 / 1 day(s)

Recording data Patient details Anamneses Patient diary Recording details Attachments

Download
 Include all attachments

Cancel Save

Recording data

<input type="checkbox"/>	Day No.	Start date of file	Duration	Status	Quality	Notes
<input type="checkbox"/>	1/1	23.05.2021 19:53	08h 00m	● Open	Poor	Add notes

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< Back **Bittium** Dashboard Recordings EN Sales Technician Technician

Patient ID: PID020c5_005914 Bittium HSAT

Organization	Evismo test center	Created	07.09.2021 15:04	Last data transfer	No date
Status	○ Rejected	Started	No date	Last update	22.09.2021 14:37
Assignment	None	Ready for analysis	No date	Duration	

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Reject recording

Patient ID: **PID023d3**

The recorder 0000780 is currently in use in the selected recording. It will be released when the recording is rejected. Are you sure you want to reject the recording?

This action cannot be undone.

Why do you want to reject the recording?

- Corrupted recording data
- Not enough analyzable data due to noisy signal or unattached recorder
- Not enough sleep to write a statement
- Other

Cancel

Reject

Delete recording

Patient ID: **PID023d3**

The recorder 0000780 is currently in use in the selected recording. It will be released when the recording is deleted. Are you sure you want to delete the recording?

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Organization	Created	Last data transfer
Bittium HSAT	20.12.2021 14:06	20.12.2021 14:58

Status	Started	Last update
Rejected	20.12.2021 14:09	21.12.2021 15:11

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The screenshot shows the Bittium patient recording interface. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings' tabs. The user is logged in as 'Sales Technician' in 'EN' language. The patient ID is 'PID0a74a' and the device is 'Bittium HSAT'. Below this, there is a table with patient details:

Organization	Sleep test center	Created	16.12.2021 15:27	Last data transfer	16.12.2021 16:24
Status	Closed	Started	16.12.2021 15:36	Last update	21.12.2021 13:58
Assignment	None	Ready for analysis	17.12.2021 14:24	Duration	2 / 2 day(s)

Below the table, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' button is highlighted with a red box, and there is an option to 'Include all attachments'. There are 'Cancel' and 'Save' buttons.

The 'Recording data' section shows a table with columns: Day No., Start date of file, Duration, Status, Quality, and Notes. Two rows are visible:

Day No.	Start date of file	Duration	Status	Quality	Notes
1/2	16.12.2021 15:36	00h 29m	Open	Poor	Add notes
2/2	17.12.2021 10:01	03h 00m	Open	Poor	Add notes

The checkbox for the second row (2/2) is checked and highlighted with a red box.

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Copyright 2024 Bittium Biosignals Oy. Kaikki oikeudet pidätetään. Tässä asiakirjassa esitetyt tiedot voivat muuttua ilman erillistä ilmoitusta. Bittium Biosignals Oy säilyttää tässä asiakirjassa esitetyn aineiston omistus- ja kaikki muut oikeudet. Tämän asiakirjan sisällön kopiointi ilman Bittium Biosignals Oy:n etukäteen antamaa kirjallista lupaa on kielletty.
Bittium Biosignals Oy