# Bittium

---

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The screenshot shows the Bittium Recordings page. At the top, there is a navigation bar with the Bittium logo, 'Dashboard', and 'Recordings' tabs. On the right, it shows 'EN' and a user profile for 'Sales Technician'. Below the navigation bar, the 'Recordings' section has a search bar, a 'Create recording' button, and filters for '10' items and 'All'. The main content is a table with the following columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. The table lists 11 recordings with various statuses (Active, Closed) and dates.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
KK02	Active	28.10.2021 15:25	28.10.2021 15:46	Sleep sales center	28.10.2021 15:23	Bittium HSAT	0000568	Sales diag group 1	Download
Pat123456	Closed	19.10.2021 11:41	19.10.2021 12:12	Sleep sales center	19.10.2021 11:40	Bittium HSAT	0000673	Sales diag group 1	Download, Refresh
PID0000b	Closed	18.10.2021 17:48	18.10.2021 18:16	Sleep sales center	18.10.2021 17:47	Bittium HSAT	0000673	Sales diag group 1	Download, Refresh
PID0000a	Closed	18.10.2021 16:19	18.10.2021 16:15	Sleep sales center	18.10.2021 16:18	Bittium HSAT	0000536	Sales diag group 1	Download, Refresh
PID00009	Active	16.10.2021 08:30	09.11.2021 10:16	Sleep sales center	16.10.2021 08:29	Bittium HSAT	0000536	Sales diag group 1	Download
PID00008	Active	16.10.2021 08:25	16.10.2021 09:04	Sleep sales center	16.10.2021 08:25	Bittium HSAT	0000568	Sales diag group 1	Download
Test008	Closed	15.10.2021 19:01	18.10.2021 15:26	Sleep sales center	15.10.2021 19:00	Bittium HSAT	0000673	Sales diag group 1	Download, Refresh
TEST007	Closed	15.10.2021 18:46	15.10.2021 19:14	Sleep sales center	15.10.2021 18:43	Bittium HSAT	0000673	Sales diag group 1	Download, Refresh
PID00007	Active	No date	14.10.2021 17:22	Sleep sales center	14.10.2021 17:18	Bittium HSAT	1234567	Sales diag group 1	Download
23_45_08	Closed	01.10.2021 16:17	14.10.2021 17:19	Sleep sales center	01.10.2021 16:16	Bittium HSAT	0123456	Sales diag group 1	Download, Refresh

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The screenshot shows the 'Recordings' section of the Bittium interface. At the top, there are navigation tabs for 'Dashboard' and 'Recordings', along with user information for 'Technician 1'. Below the navigation, there is a search bar and a 'Create recording' button. The main area contains a table with columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open for the 'Assignment' column of the first row, showing a search bar and a list of groups: 'Cardioscope Group', 'Diagnostic group', 'Diagnostic group (autoreports)', 'HRV group (autoreports)', and 'HSAT Analysis' (which is selected). Below the groups, it lists 'Users in HSAT Analysis' with 'Dr. Hanna HSAT'.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
testPDF	Closed	23.01.2024 15:14	25.01.2024 13:41	ServiceProvider	23.01.2024 15:14	Bittium	0000683	HSAT Analysis	[Download]
HSAT_ECG	Ended	16.01.2024 16:13	16.01.2024 16:17	DemoClinic1	16.01.2024 16:13	Bittium			[Download]
HSAT_2_RIP_Belts	Active	16.01.2024 16:11	25.01.2024 10:10	DemoClinic1	16.01.2024 16:11	Bittium			[Download]
PID00314	Ended	14.12.2023 15:06	19.12.2023 13:09	DemoClinic1	14.12.2023 15:05	Bittium			[Download]
PID002f3	Closed	22.11.2023 11:37	22.11.2023 11:41	DemoClinic1	22.11.2023 11:37	Bittium			[Download]

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The screenshot shows the 'Recordings' section of the Bittium interface. At the top, there are navigation tabs for 'Dashboard' and 'Recordings', along with user information for 'Sales Technician'. Below the navigation, there is a search bar and a 'Create recording' button. The main area contains a table with columns: Patient ID, Status, Last data transfer, Last update, Organization, Created, Type, Device, Assignment, and Action. A dropdown menu is open for the 'Action' column of the first row, showing a 'Download' button and a list of items: 'Recording data', 'Recording data 1 day(s)', 'Reports', '6169a6099ec5e8000acd5a02\_statement.pdf', and 'Other documents', 'recording-information.pdf'.

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
Test008	Closed	15.10.2021 19:01	18.10.2021 15:26	Sleep sales center	15.10.2021 19:00	Bittium HSAT	0000673	Sales diag group 1	[Download]
TEST007	Closed	15.10.2021 18:46	15.10.2021 19:14	Sleep sales center	15.10.2021 18:43	Bittium HSAT	0000673		[Download]
PID0000b	Closed	18.10.2021 17:48	18.10.2021 18:16	Sleep sales center	18.10.2021 17:47	Bittium HSAT	0000673		[Download]
PID0000a	Closed	18.10.2021 16:19	18.10.2021 18:15	Sleep sales center	18.10.2021 16:18	Bittium HSAT	0000576		[Download]

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**Patient ID: PID0a74a** Bittium HSAT Approved

Organization	Sleep sales center	Created	19.10.2021 11:40	Last data transfer	19.10.2021 11:41
Status	<span style="color: red;">●</span> Closed	Started	18.10.2021 23:30	Last update	19.10.2021 12:12
Assignment	Sales diag group 1 ▾	Ready for analysis	No date	Duration	1 / 1 day(s)

Recording data   **Patient details**   Anamneses   Patient diary   Recording details   Attachments

---

**Patient details**

Patient ID	Pat123456	Age (years)	55
	<small>Patient ID must be at least 4 characters long and lower/uppercase letters, numbers and underscores are allowed. If Patient ID is left blank the system will generate it (for example PID0000A).</small>	Height (cm)	175
		Weight (kg)	80
		BMI	26.12
		Gender	Female ▾

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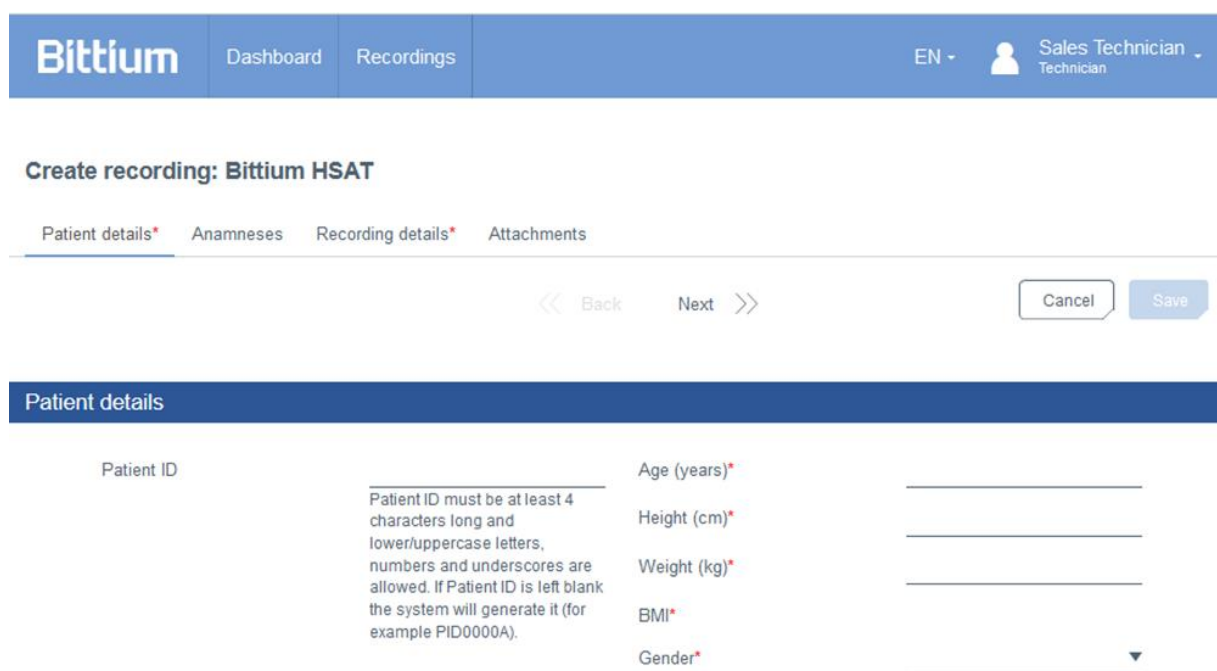
A screenshot of the Bittium MedicalSuite Center interface. At the top, there is a navigation bar with 'Bittium', 'Dashboard', and 'Recordings' tabs. On the right, it shows 'EN' and a user profile for 'Sales Technician'. Below this, the main heading is 'Create recording: Bittium HSAT'. There are four tabs: 'Patient details\*' (active), 'Anamneses', 'Recording details\*', and 'Attachments'. Navigation buttons for 'Back' and 'Next' are present, along with 'Cancel' and 'Save' buttons. The 'Patient details' section contains several input fields: 'Patient ID' with a note that it must be at least 4 characters long and allow lowercase/uppercase letters, numbers, and underscores; 'Age (years)\*', 'Height (cm)\*', 'Weight (kg)\*', 'BMI\*', and 'Gender\*'. Each field has a corresponding input line.

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The screenshot displays the Bittium web application interface for creating a recording. The top navigation bar includes the Bittium logo, 'Dashboard', 'Recordings', and user information for 'Sales Technician'. The main heading is 'Create recording: Bittium HSAT'. Below this, there are tabs for 'Patient details', 'Anamneses', 'Recording details', and 'Attachments'. Navigation controls include 'Back', 'Next', 'Cancel', and 'Save' buttons. The 'Anamneses' section is active, showing two main categories: 'Medication' and 'Medical history'. The 'Medication' section has a text input field for 'Medication' and radio buttons for 'Medication during the recording' (No/Yes). The 'Medical history' section includes a question about family members and a list of medical conditions with 'No/Yes' radio buttons. The 'Life' section includes questions about smoking, professional driving, and driving license class.

**Medication**

Medication:

Medication during the recording:  No  Yes

**Medical history**

Close family members diagnosed or treated for a sleep disorder:  No  Yes

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## Chance of dozing off or falling asleep in the following situations

Sitting and reading	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Watching TV	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting, inactive in a public place (e.g. a theatre or a meeting)	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
As a passenger in a car for an hour without a break	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Lying down to rest in the afternoon when circumstances permit	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting and talking to someone	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
Sitting quietly after a lunch without alcohol	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High
In a car, while stopped for a few minutes in the traffic	<input type="radio"/> Never	<input type="radio"/> Slight	<input type="radio"/> Moderate	<input type="radio"/> High

## Sleep history

Morning headache	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Dry mouth in the morning	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Daytime fatigue or snoring	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Daytime napping	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Heartburn	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Loss of concentration	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Memory loss	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Cramping at night	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Difficulties falling asleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Breathing breaks while sleeping	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Pain during night	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Snoring or snorting during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Waking up during the night or restless sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
A strong urge to move your feet or an uncomfortable sensation in your feet related to this urge during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
Urinary urgency during sleep	<input type="radio"/> Never	<input type="radio"/> Rarely	<input type="radio"/> Sometimes	<input type="radio"/> Often	<input type="radio"/> Almost always
If yes, how many times per night?	<input type="radio"/> Once	<input type="radio"/> Twice	<input type="radio"/> More often		
Average amount of sleep	_____ Hours				

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**Create recording: Bittium HSAT**

Patient details\* Anamneses **Recording details** Attachments

<< Back Next >>

Cancel

Save

**Recording details**

Organization	ServiceProvider	Referring physician
Requested report language	English	
Reason for recording	<input type="text"/>	

**Device information**

Recorder S/N (21) 19 XXXXXXX

See the side of the device. **Please add numbers after "19".**

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## Create recording: Bittium HSAT

Patient details Anamneses Recording details\* Attachments

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Patient ID: **PID0a74a** Bittium HSAT

Ready for analysis |

Organization	Sleep test center	Created	29.12.2021 12:06	Last data transfer	29.12.2021 12:20
Status	<span style="color: green;">●</span> Active	Started	09.09.2021 23:30	Last update	29.12.2021 12:21
Assignment	None	Ready for analysis	No date	Duration	2 / 2 day(s)

[Recording data](#)
[Patient details](#)
[Anamneses](#)
[Patient diary](#)
[Recording details](#)
[Attachments](#)

## Patient diary - 09.09.2021, 23:30 (06h 59m)

Please, ask the patient to answer the following questions about the recording.

When did you go to bed?	<input type="text" value="hh:mm"/>	In your estimation, when did you fall asleep?	<input type="text" value="hh:mm"/>
What time did you wake up in the morning?	<input type="text" value="hh:mm"/>	In your estimation, how many hours did you sleep?	<input type="text" value="hh:mm"/>
How did you sleep?	<input type="text"/>	Did you get out of bed during the recording?	<input type="text"/>
How did you sleep compared to usual?	<input type="text"/>		

Other comments

If the patient woke up during the recording, please write down the events.

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The screenshot displays the Bittium interface for recording data. At the top, a summary table shows the recording details:

Organization	Clinic A	Created	01.04.2019 15:34	Last data transfer	13.04.2019 12:00
Status	Preparing	Started	10.04.2019 12:00	Last update	13.04.2019 12:00
Assignment	None	Ended	No date	Duration	2 days

Below this, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' button is visible with an option to 'Include all attachments'. A 'Cancel' button and a 'Save' button are also present.

The main section is titled 'Recording data' and contains a table with columns: Day no., Start time, Duration, Status, Quality, and Notes. The table has two rows:

Day no.	Start time	Duration	Status	Quality	Notes
1 / 2	10.04.2019 22:00	08h 05m	Open	Fair	Add notes
2 / 2	11.04.2019 22:00	07h 50m	Open	Good	Add note

An 'Add file or drag here' button is located above the table. A red box highlights the 'Preparing' status in the summary table and the 'Preparing data - 2 / 2' entry in the table.

An 'UPLOADS' window is open in the bottom right corner, showing the patient ID: 123455677888. It lists two items: 'Uploaded 123455677886.edf 123455677887.edf' with a green checkmark, and 'Preparing data - 2 / 2' with a blue circular arrow icon.

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The screenshot shows the Bittium MedicalSuite interface for a patient with ID PID0a74a. The top navigation bar includes 'Back', 'Bittium', 'Dashboard', 'Recordings', 'EN', and a user profile for 'Sales Technician'. The patient's status is 'Ready for analysis'. Below this, a table provides details for the recording:

Organization	Sleep test center	Created	29.12.2021 11:22	Last data transfer	29.12.2021 11:32
Status	Active	Started	16.12.2021 02:44	Last update	29.12.2021 11:33
Assignment	None	Ready for analysis	No date	Duration	2 / 2 day(s)

Below the table are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' button is present with an option to 'Include all attachments'. A 'Recording data' section shows a table with one entry:

Day No.	Start date of file	Duration	Status	Quality	Notes
1/1	16.12.2021 00:44	08h 10m	Open	Good	Add notes

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The screenshot displays the 'Recording data' section of the Bittium Respiro Analyst interface. At the top, there are navigation tabs: 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. Below the tabs, there are options for 'Download' and 'Include all attachments', along with 'Cancel' and 'Save' buttons. The main area contains a table with the following columns: 'Day No.', 'Start date of file', 'Duration', 'Status', 'Quality', and 'Notes'. Three rows of data are visible:

Day No.	Start date of file	Duration	Status	Quality	Notes
1/3	14.12.2021 23:42	04h 30m	Done	Fair	Add notes
2/3	14.12.2021 23:42	05h 00m	Open	Good	Add notes
3/3	14.12.2021 23:42	00h 15m	Unusable	Poor	Marked as "Unusable" <ul style="list-style-type: none"><li>Corrupted recording data</li><li>Not enough analyzable data due to noisy signal or unattached recorder</li><li>Not enough sleep to write a statement</li><li>Other (trs)</li></ul>

A dropdown menu is open for the 'Status' column of the third row, showing options: 'Open', 'Pending', 'Done', 'Unusable', and 'Unusable' (highlighted). A red box highlights the dropdown menu.

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Unusable recording data

Patient ID: [patient\_id], 10/04/2019, 22:00 (08h 20m)

Why the recording data of this night is unusable?

Corrupted recording data

Not enough analyzable data due to noisy signal or unattached recorder

Not enough sleep to write the statement

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The screenshot displays the Bittium MedicalSuite interface for a patient recording. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings' tabs. The user is identified as 'Sales Technician'. The patient ID is 'PID0a74a'. A 'Ready for analysis' button is highlighted with a red box. Below this, a table shows recording details: Organization (Sleep test clinic A), Created (24.03.2022 15:55), Last data transfer (24.03.2022 15:58), Status (Active), Started (14.12.2021 23:42), Last update (05.05.2022 13:15), Assignment (None), Ready for analysis (No date), and Duration (2 / 2 day(s)). A dropdown menu for the status is also highlighted with a red box, showing 'Active' and 'Ended' options. Below the table, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' button and a checkbox for 'Include all attachments' are visible. At the bottom, there is a 'Recording data' section with a table containing one row of data: Day No. 1/2, Start date of file 14.12.2021 23:42, Duration 04h 30m, Status Open, Quality Fair, and Notes Add notes.

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Ready for analysis

Patient ID **PID0a870\_013696**

Are you sure this recording is ready for analysis?

- Please make sure the patient diary is filled or it is attached to "Other documents" section
- You will no longer be able to upload new recording data to it.

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## Diagnostic report needed

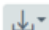
Recording	Assignment	Ready for analysis	Duration	
● PID0a74a	Test diag group 1 ▾	24.11.2021 09:10	1 day(s)	Analyze 

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The screenshot shows the Bittium interface for recording management. At the top, there's a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is logged in as 'Technician 1'. The main content area shows details for 'Patient ID: PID00298' with a 'Ready for analysis' status. Below this is a table of recording details:

Organization	DemoClinic1	Created	30.05.2023 13:56	Last data transfer	30.05.2023 14:01
Status	Active	Started	23.11.2021 23:42	Last update	25.01.2024 14:08
Assignment	HSAT Analysis	Ready for analysis	No date	Duration	1 / 1 day(s)

Below the table, there's a dropdown menu for 'Assignment' which is currently set to 'HSAT Analysis'. The dropdown is open, showing a search bar and two sections: 'Groups' and 'Users in HSAT Analysis'. The 'Groups' section includes 'Cardioscope Group', 'Diagnostic group', 'Diagnostic group (autoreports)', 'HRV group (autoreports)', and 'HSAT Analysis' (which is selected with a checkmark). The 'Users in HSAT Analysis' section includes 'Dr. Hanna HSAT' (which is also selected).

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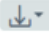
## Diagnostic report needed

Recording	Assignment	Ready for analysis	Duration	
● PatientZero	Test diag group 1 ▾	24.11.2021 09:10	1 day(s)	Analyze 

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1 of 3 Automatic Zoom

### Patient

Patient ID:	23-30-48	Height:	185cm	Weight:	85kg
BMI:	24.8	Age:	58 years	Gender:	Male

### Summary

Apnea-hypopnea index: 8.7/h  
(Supine: 15.4/h, Non-supine: 2.1/h)

Oxygen Desaturation Index (ODI3): 8.7/h

AHI

Normal 5 Mild 15 Moderate 30 Severe

ODI3

Recording date	15/10/2021	Recording: TRT	08h 36m (23:30 - 08:07)
Evaluation period	08h 16m (23:43 - 08:00)		

Apnea-hypopnea index (AHI) displayed here is the total number of respiratory events (Apneas + Hypopneas) divided by Evaluation period. Evaluation period was determined based on measured signals and patient diary

### Statement

#### Findings

Nearly all occurrences of respiratory events are in the supine position. The recording was technically successful and of high quality.

#### Conclusion

The recording findings are consistent with mild obstructive sleep apnea. Avoiding sleeping in a supine position (positional therapy), for example by placing a tennis ball in the back of the pajamas or wearing a sleep apnea belt, might be sufficient treatment. It is recommended to confirm the treatment outcome with a new recording.

23/02/2022

Dr. Sleep Apnea

*This report is the consulting doctor's opinion based on the monitoring and the available background information. The final conclusions are always made by the patient's attending doctor who has the best*

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## Accept the report

Once you accept the report, the recording will be closed. Are you sure you want to accept the report?

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Patient ID: PID0a743\_013381 Bittium HSAT

Organization	Sleep test center	Created	13.12.2021 09:29	Last data transfer	13.12.2021 09:31
Status	● Ended	Started	24.06.2021 00:00	Last update	29.12.2021 16:20
Assignment	Daniela DiagnosticUser	Ready for analysis	13.12.2021 09:44	Duration	1 / 1 day(s)

Recording data Patient details Anamneses Patient diary Recording details Attachments

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Recordings

Search: PID0a743 Create recording

Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
PID0a743	● Ended	13.12.2021 09:31	29.12.2021 16:20	Sleep test center	13.12.2021 09:29	Bittium HSAT	0000552	Daniela DiagnosticUser	📄 ⬇️

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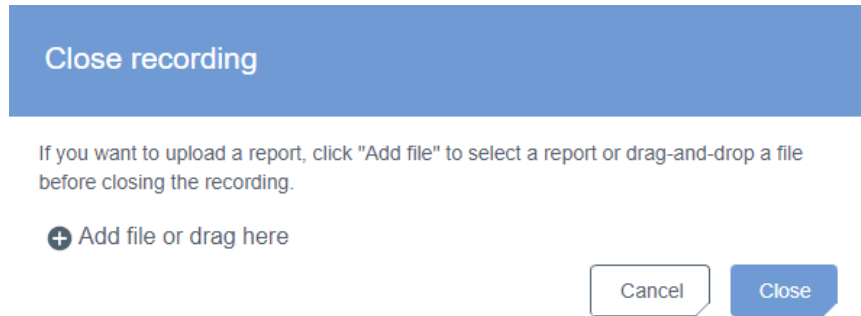
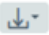
Latest reports				
Recording	Organization	Added	Approval	
● PID0a74a	DemoClinic2	07.10.2020 17:49	07.10.2020 17:54	 

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Recordings									
PID0a74a		Create recording -		10 -		All -			
Patient ID	Status	Last data transfer	Last update	Organization	Created	Type	Device	Assignment	Action
PID0a74a	● Closed	14.12.2021 15:47	17.12.2021 15:22	Sleep test center	14.12.2021 15:47	Bittium HSAT	0123456	None	 

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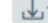
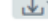
Latest reports				
Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	 
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	<b>Approve</b>	 

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EN - Sales Technician Technician

Patient ID: PID0a74a Bittium HSAT Approve

Organization	Sleep test center	Created	14.12.2021 15:47	Last data transfer	14.12.2021 15:47
Status	<span style="color: red;">●</span> Closed	Started	24.02.2021 00:00	Last update	17.12.2021 15:22
Assignment	None	Ready for analysis	14.12.2021 15:49	Duration	1 / 1 day(s)

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**Confirmation**

Are you sure you want to approve the reports of Patient ID: 01-10-56? After approval, reports will be visible for the clinic.

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< Back **Bittium** Dashboard Recordings EN Sales Technician Technician

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Organization	Sleep test center	Created	14.12.2021 15:47	Last data transfer	14.12.2021 15:47
Status	● Closed	Started	24.02.2021 00:00	Last update	31.12.2021 10:13
Assignment	None	Ready for analysis	14.12.2021 15:49	Duration	1 / 1 day(s)

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Include all attachments

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Latest reports

Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	<span style="border: 1px solid red; padding: 2px;">📄</span> <span>↓</span>
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	16.12.2021 09:48	<span>📄</span> <span>↓</span>

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## Latest reports

Recording	Organization	Updated	Approval	
● PID023d3	Sleep test center	16.12.2021 09:46 Not assigned	16.12.2021 09:48	 
● PID0a74a	Sleep test center	15.12.2021 15:16 Not assigned	16.12.2021 09:48	 

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The screenshot displays the Bittium MedicalSuite Center interface. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is logged in as 'Sales Technician'. The main content area shows 'Patient ID: PID0a870\_013696' and 'Bittium HSAT'. A table lists recording details:

Field	Value	Field	Value	Field	Value
Organization	Sleep test center	Created	21.12.2021 13:38	Last data transfer	21.12.2021 13:39
Status	Ended	Started	23.05.2021 22:53	Last update	23.12.2021 13:41
Assignment	None	Ready for analysis	21.12.2021 13:42	Duration	1 / 1 day(s)

Below the table, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' button is visible, along with a checkbox for 'Include all attachments' and 'Cancel'/'Save' buttons. A 'Recording data' table is shown below:

Day No.	Start date of file	Duration	Status	Quality	Notes
1/1	23.05.2021 19:53	08h 00m	Open	Poor	Add notes

A red box highlights the 'Close' and 'Reject' buttons in the top right corner of the recording detail view.

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Organization	Evismo test center	Created	07.09.2021 15:04	Last data transfer	No date
Status	<input type="radio"/> Rejected	Started	No date	Last update	22.09.2021 14:37
Assignment	None	Ready for analysis	No date	Duration	

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Patient ID: **PID023d3**

The recorder 0000780 is currently in use in the selected recording. It will be released when the recording is rejected. Are you sure you want to reject the recording?

**This action cannot be undone.**

Why do you want to reject the recording?

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Not enough sleep to write a statement

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Patient ID: **PID023d3**

The recorder 0000780 is currently in use in the selected recording. It will be released when the recording is deleted. Are you sure you want to delete the recording?

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The screenshot shows the Bittium patient interface. At the top, there is a navigation bar with 'Back', 'Bittium', 'Dashboard', and 'Recordings'. The user is identified as 'Sales Technician'. The patient information is 'Patient ID: PID0a74a' and 'Bittium HSAT'. Below this, a table displays recording details:

Organization	Sleep test center	Created	16.12.2021 15:27	Last data transfer	16.12.2021 16:24
Status	Closed	Started	16.12.2021 15:36	Last update	21.12.2021 13:58
Assignment	None	Ready for analysis	17.12.2021 14:24	Duration	2 / 2 day(s)

Below the table, there are tabs for 'Recording data', 'Patient details', 'Anamneses', 'Patient diary', 'Recording details', and 'Attachments'. A 'Download' dialog box is open, with a red box highlighting the 'Download' button and the 'Include all attachments' checkbox. The dialog also has 'Cancel' and 'Save' buttons.

The 'Recording data' section shows a table with the following data:

<input type="checkbox"/>	Day No.	Start date of file	Duration	Status	Quality	Notes	
<input type="checkbox"/>	1/2	16.12.2021 15:36	00h 29m	Open	Poor	Add notes	Download
<input checked="" type="checkbox"/>	2/2	17.12.2021 10:01	03h 00m	Open	Poor	Add notes	Download

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Copyright 2024 Bittium Biosignals Ltd. Tutti i diritti sono riservati. Le informazioni contenute nel presente documento sono soggette a modifica senza preavviso. Bittium Biosignals Ltd mantiene la proprietà dei materiali contenuti nel presente documento ed è titolare di tutti i diritti ad essi relativi. È vietata qualsiasi riproduzione del contenuto del presente documento senza previo consenso scritto di Bittium Biosignals Ltd.
#### **Bittium Biosignals Ltd**